



iD PANTS FITTING GUIDE



YOUR TRUSTED PARTNER
IN CONTINENCE CARE



iD PANTS

iD Pants offer an ideal solution for managing moderate to heavy urinary incontinence. iD Pants can be worn and taken off like any regular underwear. These products combine comfort and discretion due to their elasticated waist and CottonFeel backsheet.

STEP 1

Start by folding the iD Pants in half lengthways to activate the anti-leak cuffs.

STEP 2

Ensuring the sticker is at the back, step into the iD Pants and pull them up, adjusting for comfort and security.







STEP 3

Remove the iD Pants by either pulling them down or tearing the side seams.

STEP 4

Once removed, simply roll-up the pad and dispose of carefully.

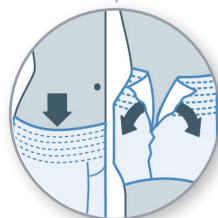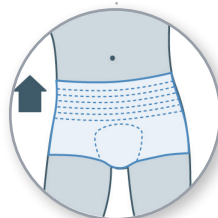



IMPORTANT ADVICE

The use of talcum powder is not recommended, oil based barrier creams may reduce the performance of the pads. If the barrier creams must be used they should be applied sparingly and only on treatable areas.

Used products should be disposed of in the appropriate manner. They cannot be flushed down the toilet.

iD PANTS

AVAILABLE WAIST SIZES:

EXTRA SMALL	50-70CM
SMALL	60-90CM
MEDIUM	80-120CM
LARGE	100-145CM
EXTRA LARGE	130-170CM

