

vivactive 

vivactive
PANTS
Fitting
Guide



Vivactive Pants

Vivactive Pants offer an ideal solution for managing light to moderate urinary incontinence. Vivactive Pants can be worn and taken off like any regular underwear.

These products combine comfort and discretion due to their elasticated waist and Cotton Feel backsheet.



Vivactive Pants


Step 1

Ensure the correct size by measuring the waist level with the umbilicus (belly button). Start by folding the pull up pants in half lengthways to activate the anti-leak cuffs.




Step 2

Ensuring the sticker is at the back, step into the pull up pants and pull them up, adjusting for comfort and security.

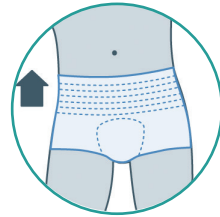

Step 3

Remove the pull up pants by either pulling them down or tearing the side seams.

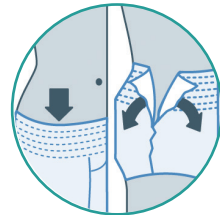

Step 4

Once removed, simply roll-up the pad and dispose of as per local policy.



Vivactive Pants

Important Advice

Oil based barrier creams may reduce the performance of the pads. If barrier creams must be used, they should be applied sparingly and only on treatable areas.

Used products should be disposed of in the appropriate manner, they cannot be flushed down the toilet.

Vivactive Ltd

Vivactive House
Unit 3 Hollinswood Court
Stafford Park 1
Telford
TF3 3DE