




FLEX


Attends Flex are a range of all-in-one belted products with a flexible fixation system which allows the belt to be fitted snugly around the waist. Designed to manage heavy urinary and / or faecal incontinence.




Fitting Guide

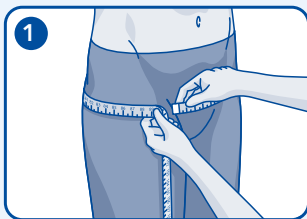


1 Measure the hip and waist and choose the larger of these two to select the correct product size.

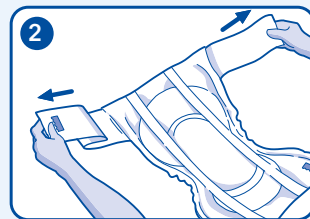

2 Open the product fully and position the end with the belts to the back. Make sure each belt is also fully opened and that the back sheet is on the outside.

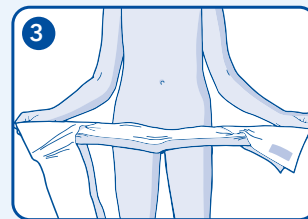


3 Gently fold and cup the product lengthways to ensure that the leakage barriers stand up.

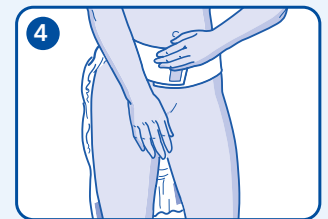


4 Position the belt high up around the waist and secure the product in place by pressing the blue hook firmly on to the belt.



5 Gently pull the product through from back to front, keep it folded to ensure the leakage barriers stand up, and pull up high into the leg creases of the groin to give a snug fit.

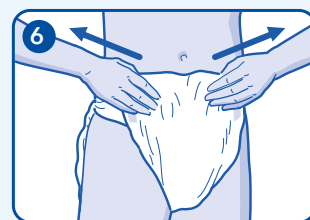

6 Smooth out the product and pull it straight up to the waistband. Secure the product in place by pressing the blue hooks firmly on to the waistband.

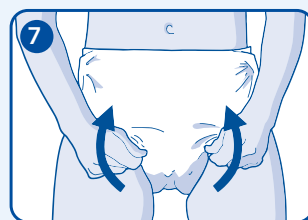

7 If necessary readjust the product for a secure and comfortable fit.

FLEX

Hints and Tips

Selection

- Choose the correct product size according to the hip and waist size and take whichever is the largest.
- Choose the absorbency based on the volume of urine passed. Start by choosing a lower absorbency number before moving to a higher absorbency.

Fitting

- Gently fold and cup the product lengthways. Remember to pull the product straight up before attaching the hooks to the waistband. If necessary readjust the product.
- Keep the belt at the back when fitting the product - you then know it is the correct way round.
- Use the wetness indicator to centre the product.
- To secure the product in place, press the hook firmly on to the belt.
- If the product is fitted when the user is laying down follow the same sequence as for fitting the product in a standing position.

Changing

- Check the wetness indicator - if the wetness indicator has disappeared over half the length of the product (60%) or if it contains faeces the product requires changing. If not the product can be reapplied.
- Clean and dry products can be resealed and reused after toileting.
- Remove soiled products from the rear where possible.
- Clean skin as per your local policy. Clean from front to back.
- Only use prescribed creams and apply as per the manufacturer's instructions.

Leakage

- Check all aspects of fitting and assessment.
- Check that you have the correct size.

Disposal

- Roll up the product and dispose of as per your local authority or NHS waste policy.
- Do not flush the product down the toilet.



Skin tolerability is dermatologically approved by SGS proderm Institute for Applied Dermatological Research